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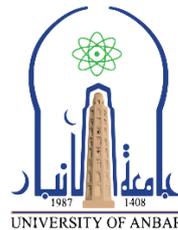
UNIVERSITY OF ANBAR

Body Fluid Exam : Lab NO.6

Methods Of Identification Of Some Parasites in Body Fluids

Laboratory testing can be performed on many types of fluids from the body other than blood. Often, these fluids are tested instead of blood because they can give more direct answers to what may be going on in a particular part of the body.

Some body fluids that may be tested include:

- Blood
- Urinalysis
- CSF Analysis
- Semen Analysis
- Pericardial Fluid Analysis
- Peritoneal Fluid Analysis

Urine Examinations

In Medical Parasitology the main aim of a urine examination is to find *Schistosoma haematobium* eggs , *Trichomonas*.

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The specimen should be collect when the chance to find eggs is the highest (between 10 a.m. and 2 p.m.). One should examine as much urine as possible to detect light infections (several specimen from different days would be optimal).

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There are two main methods for the detection of *S. haematobium* eggs:

1. Sedimentation

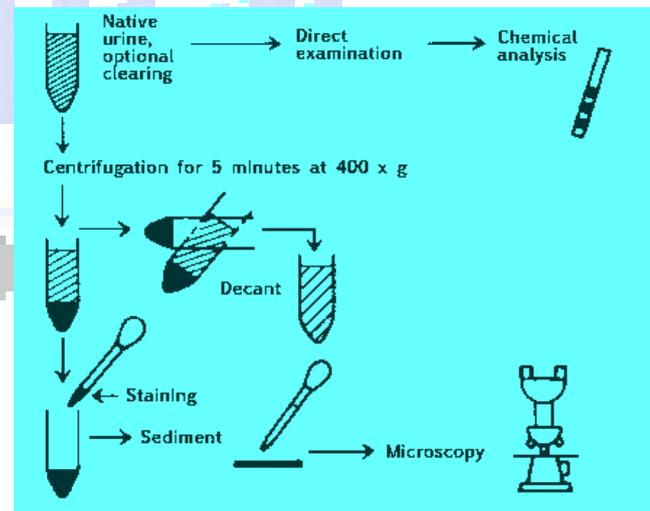
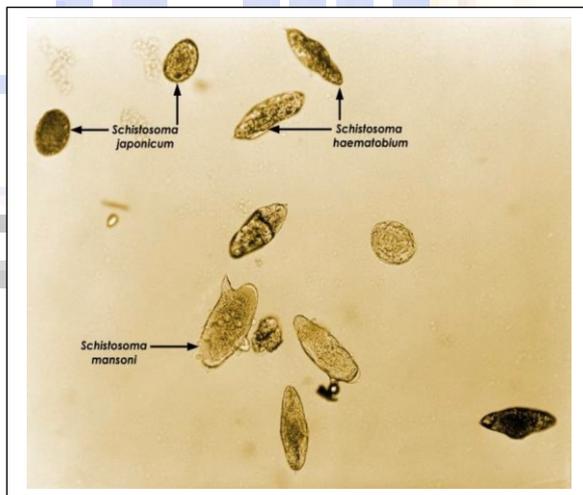
Let the fresh urine sample (shake before pouring) to sediment for 1 hour in a conical urine flask. Withdraw the sediment into a centrifuge tube and spin for not more than 2 minutes at 2000 g. Examine the sediment at low magnification for the presence of ova.

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2. Filtration

Place a polycarbonate or nylon fiber filter (12-20 μm pore size) in a filter holder. Agitate the urine and fill a syringe (10-20 ml). Attach the filter holder and expel the urine. Remove the filter with forceps on a microscopic slide. Examine the whole filter for eggs after staining with Lugol's iodine.

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